



# Lab help

## Cosmetic & Full Mouth Checklist for Success

If you have any questions or need assistance you can contact us at 800-336-3053 or visit our website and click on the icon as shown to the right, which you can find along the bottom right hand corner of our website at [burbankdental.com](http://burbankdental.com). Just request Lab Help and we will be glad to assist you.



Also, check out our Resources section when visiting our website.

 **BURBANK**  
DENTAL LAB  
800-336-3053

## 1 First Appointment

- 1 Pre-Op Models Photos
  - 1 Full face smile (forehead to chin), Lip at rest close-up, Smile close-up
  - 2 Incisal Shade Picture - (Shade tabs(s) incisal edge to incisal edge on same vertical plane.)
  - 3 Over All shade Picture (Shade tab next to target shade tooth, oriented body-body, incisal to incisal)



## 2 Proposed Plan

- 1 Steps
  - 1 Quantity, tooth #'s, and pontics
  - 2 Tissue or other augmentations planned
  - 3 Staging plan
  - 4 VDO change (quantify) or remain as current
  - 5 Type of restorative material(s)
  - 6 Patient concerns and goals for case
  - 7 Any other goals or concerns

### 2 Second Appointment

Lab Delivers: Diagnostic Wax-Up, Plan Verification Report, Putty Stent of Wax-Up, Prep Guide Model & Prep-Stent




### 3 Second Appointment: Send to Lab

Rx for Dura-Temps & VDO Bite

## 3 Third Appointment

- 1 Lab Delivers: DuraTemp
  - 3 "Sections" staging of preparation & bite records
  - 2a If no VDO change
    - 1 Prep single quadrant take hard bite - Luxabite®
    - 2 Prep second quadrant take hard bite - Luxabite®
    - 3 3rd and final quadrant place previous hard bites and take 3rd hard bite

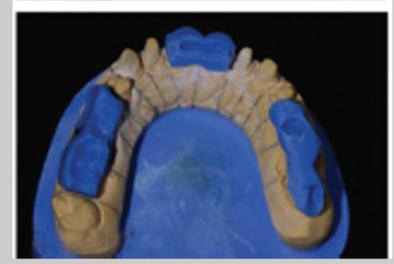



# Lab help

## Cosmetic & Full Mouth Checklist for Success

Page 2

### 3 Third Appointment Cont.

- 2b** If VDO change
- 1 Prep single section, try-in pre-made lab provisionals., reline - Check VDO Relationship.
  - 2 Repeat for 2nd and 3rd section.
  - 3 Take 3 bites each with 2 provisional sections in place. (eg. Anterior bite with both posterior section provisionals in place.)
- 2c** If both arches are being restored
- 1 Prep both maxillary & mandibular anterior sections. Try-in & reline DuraTemps, verify VDO and occlusal relationship.
  - 2 Prep one posterior section, both maxillary and mandibular. *With anterior provisionals in place.* Try-in & reline DuraTemps, verify VDO and occlusal relationship.  
  
Repeat for contra-lateral section.
  - 3 Take 3 bites for EACH section. (With provisionals seated on other 2 sections.)
    - a. Prep to prep.
    - b. (2) Prep to opposing temps.
  - 4 Check All 3 Prep to Prep bites. All 3 should fit easily and passively. If not, find the 2 that do and retake 3rd.

### 4 Fabrication of Final Restorations

- 1  Send Rx, Impressions & Bites to Lab
  - 1 Send to lab:
    - Full arch prep impressions
    - Impressions of Provisionals
    - All bites in labeled baggies
  - 2 **Fourth Appointment**  
Provisional Approval & Records
    - 1 Photos & models of approved provisionals
    - 2  Send Rx, Provisional Models and Photos to Lab
- 3  Lab Delivers: Bisque bake for try-in.
- 4 **Fifth Appointment:**  
Try-in and adjust bisque bake.  
  
 Rx for final finish of restorations

### 5 Sixth Appointment Delivery of finished restorations

- 1  Lab Delivers: Final restorations
- 2 Delivery of restorations  
  
  
